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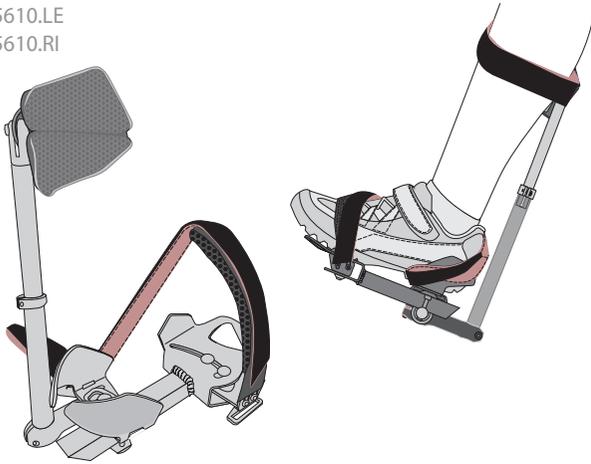
Indications for use

These pedals are intended for use by riders with foot deformities and/or motor disorders.

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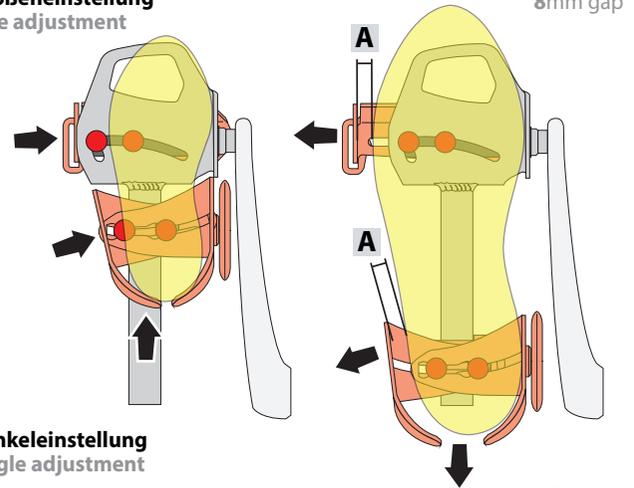
Prior to riding with these pedals for the first time, be sure to ask your doctor or physical therapist whether the use of the pedals or cycling in general poses any risks to your health. All installation work and adjustments must be performed by a qualified professional. Consult your authorized dealer. Do not make any modifications to the pedals. The pedals may not be used in any manner that adversely affects the safety of the vehicle. The pedals can only be used for multi-track vehicles (e.g. trikes) or for the front rider of the semi-recumbent tandem Pino.

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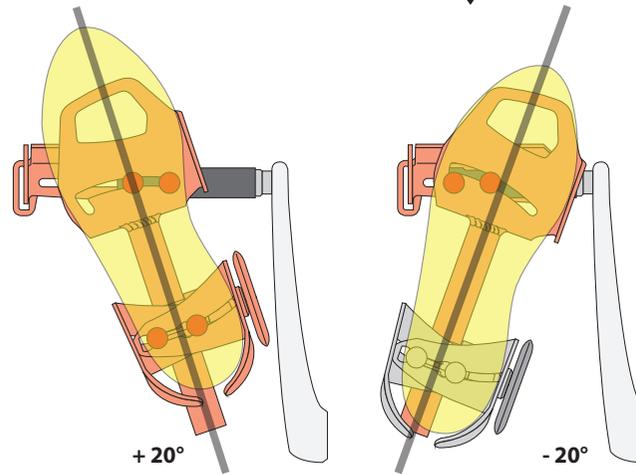
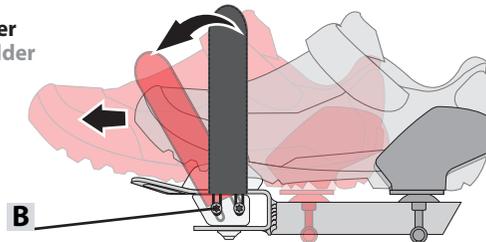
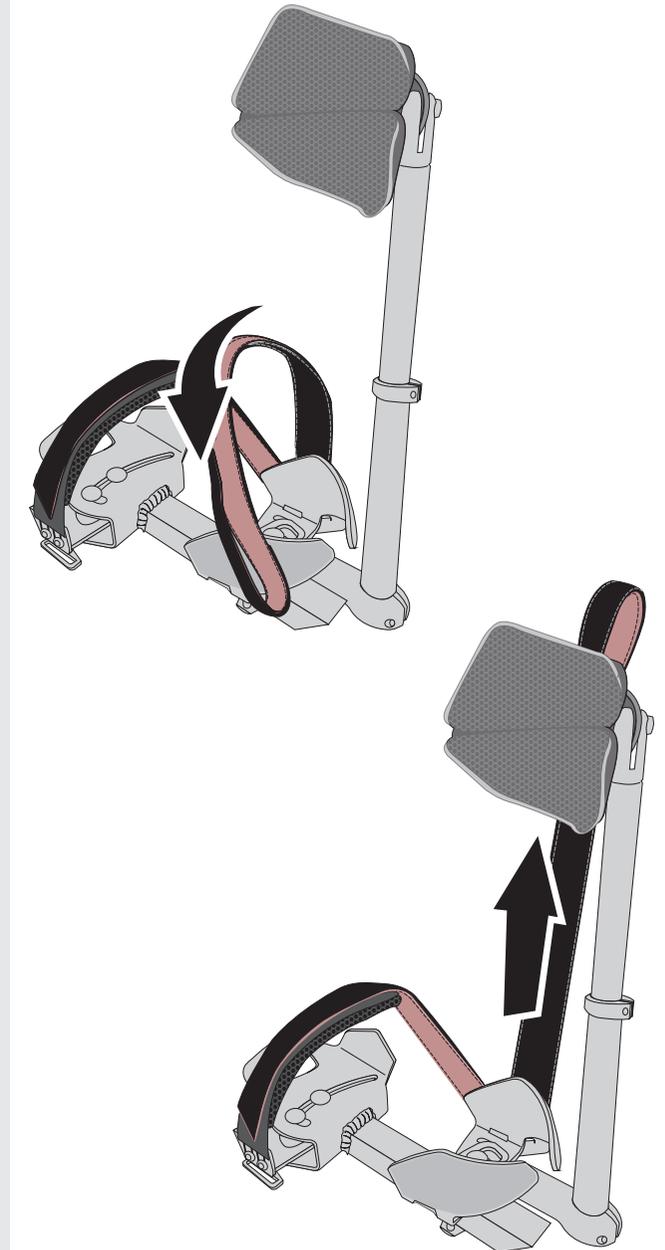
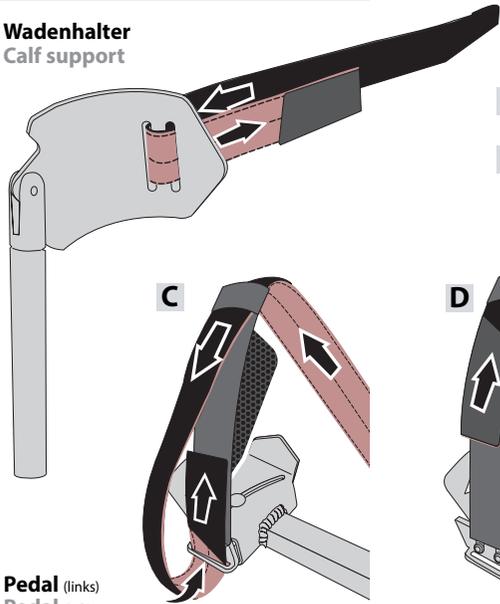
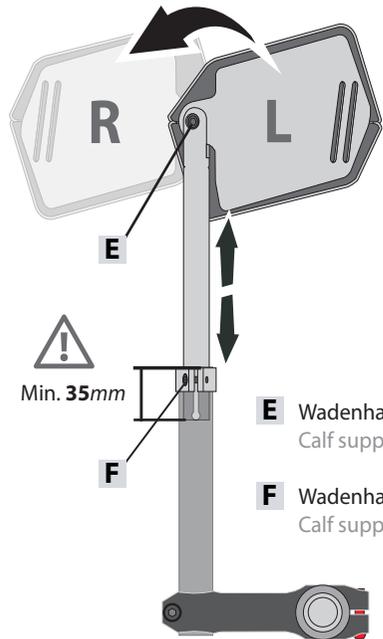
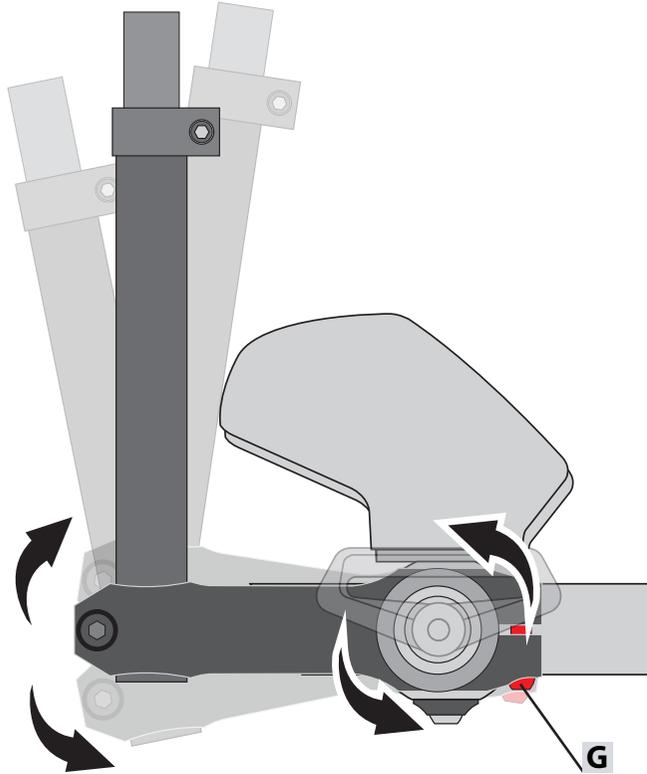
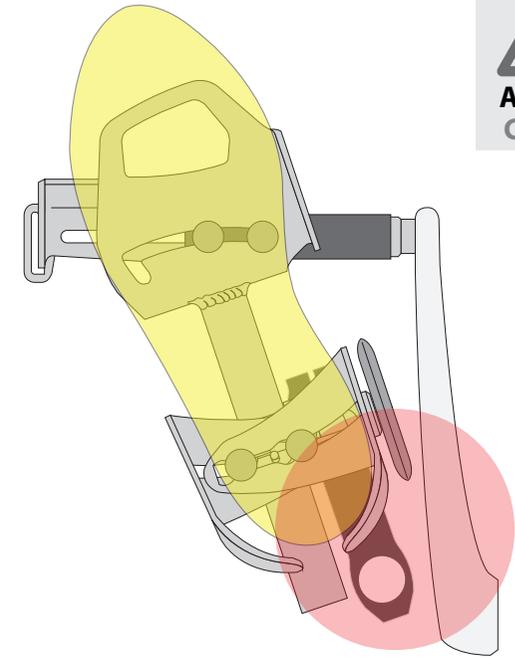
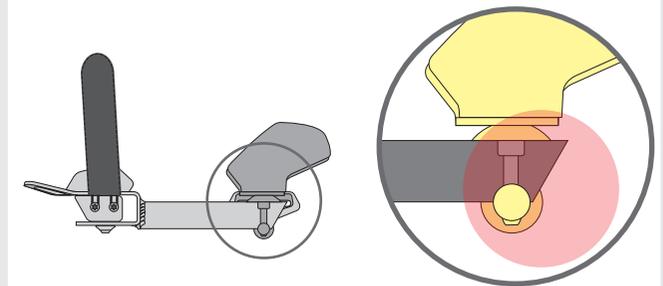
From a medical perspective, movement is important for healthy joints. This is why the mounting mechanism for the calf support uses a flexible elastomer spring that allows movement but returns the support to the middle position for easier handling. If desired, the calf support can also be set to a fixed position by tightening the bolt **G**.

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